## Instructions for use Breastmilk Bottles

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## Bruksanvisning Melkeflasker

## Breastmilk Bottles

#### Ideal to collect, store and freeze breastmilk

#### Cleaning and Hygiene

##### Important

- Use drinking water quality for any of the following described cleaning procedures.
- Clean parts immediately after use to avoid dry up of milk residues and to prevent from growth of bacteria.

##### Prior to first use and once daily

Disassemble, wash and rinse all parts. Place parts in a saucepan of sufficient capacity. Fill with cold water until the parts are well covered. Optionally, a tea spoon of citric acid may be added to avoid a build-up of limescale. Heat it up and allow to boil for 5 minutes.

Alternatively, the disassembled parts can be sanitized in Medela Quick Clean Microwave Bags or in top rack of dishwasher or steam-sterilized.

##### After each use

Disassemble and wash all parts in warm, soapy water. Rinse in clear water and allow parts to air-dry on a clean towel.

##### Caution

- All parts which are not in use must be kept out of children's reach.
- Avoid touching the inside of bottles and lids.

### Storing / thawing breastmilk

(For Healthy Term Babies)

Freshly expressed breastmilk	
Room Temperature	Do not leave at room temperature
Refrigerator	3 – 5 days at approx. 4° C
Refrigerator Freezer Compartment	6 months at approx. –16° C
Deep Freezer	12 months at approx. –18° C
Thawed breastmilk (previously frozen)	
Room Temperature	Do not store
Refrigerator	10 hours
Refrigerator Freezer Compartment	Never refreeze
Deep Freezer	Never refreeze

- When freezing do not fill bottles more than 3/4 full to allow space for expansion.
- Label bottles with the date of pumping.
- Do not store breastmilk in the door of the refrigerator.
- In order to preserve the components of the breastmilk, thaw in the refrigerator overnight or by holding the bottle under running warm water (max. 37° C).
- Gently shake the bottle to blend any fat that has separated.
- Do not thaw frozen breastmilk in a microwave oven or in a pan of boiling water (danger of burning).

Material:	Polypropylene
Disposal:	According to local regulations

Does not contain Bisphenol-A, DEHP (Phthalate) or Latex.

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