Mii™

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TIRE-LAIT MANUEL Mii™

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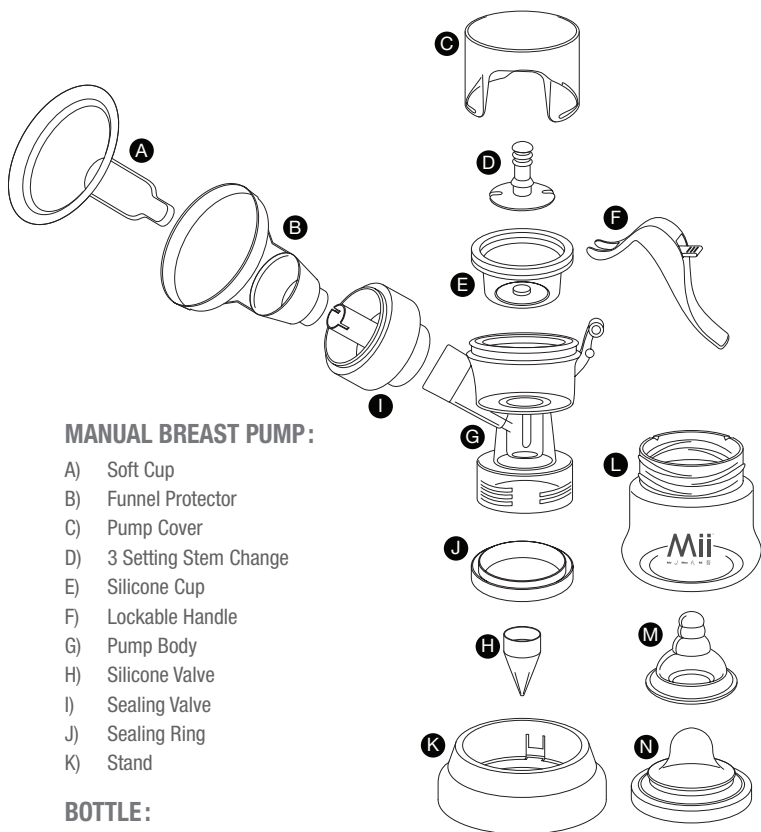
INTRODUCTION

- Breast milk is the best possible source of nutrition a mother can provide her baby with, because it contains special anti-bodies that strengthen baby's immune system.
- However, work and travel demands make breastfeeding a challenging task for some mothers.
- Mii™ Manual Breast Pump was created in order to make this precious source of nutrition more available to baby without the interference of everyday demands.
- The pump functions quietly and effectively for extracting milk at anytime.
- The soft cup is gentle to help mother express breast milk comfortably, for longer periods of time.
- The breast pump comes with 3 suction settings so you can find the suction strength that most closely simulates baby's suction strength.

DETAILED PARTS LIST

MANUAL BREAST PUMP:

- A) Soft Cup
- B) Funnel Protector
- C) Pump Cover
- D) 3 Setting Stem Change
- E) Silicone Cup
- F) Lockable Handle
- G) Pump Body
- H) Silicone Valve
- I) Sealing Valve
- J) Sealing Ring
- K) Stand

BOTTLE:

- L) Mii Storage Container
- M) Flutter Flex Teat
- N) Silicone Sealing Disc

ASSEMBLY

- Before assembly, please make sure all parts are separated and clean:
- Insert the silicone valve (H) into the pump body (G) from underneath.
- Place the sealing ring (I), flat side first, inside the pump body (G) from underneath.
- Place the pump body (G) onto the Mii™ storage container (L) and gently tighten until secure.
- **DO NOT OVERTIGHTEN!**
- Take the 3 setting stem (E) and insert it through the silicone cup (D) until secure in the notch.
- Place the silicone cup (D) into the pump body (G) through the top.
- Press down alongside the rim to secure the seal.
- Place the forked end of the lockable handle (F) into the 3 setting stem (E) under the desired notch and click into place.
- To lock the handle, make sure the lock switch is up, push handle down then push the switch down to lock in place.
- Insert the soft cup (A) into the funnel protector (B) until secure.
- Firmly place the sealing valve (I) into the end of the soft cup (A) so that the seal is flush.
- Place the soft cup (A), sealing valve (I) end first, into the pump body (G).
- Place the pump cover (C) on top of the pump body (G) and place the Mii bottle (L) into the stand (K) for stability.

CLEANING & STERILISATION

- Please follow the cleaning procedures before using the breast pump for the first time and each time after.
- Prior to first use, hand wash with soap and hot water, rinse and dry well.
- After first use, hand washing or dish washing is recommended for cleaning.
- Disassemble the pump into all of the separate parts before placing in the dishwasher or hand.
- Do not use any anti-bacterial cleaners, solvents, or abrasive detergents.
- Do not put feeding parts on surfaces that have been exposed to anti-bacterial cleaners.
- Inspect all items prior to reuse, and be sure to replace at first sign of weakness or damage.
- Products are dishwasher safe to 60°C (140°F) water temperature, but use top rack only.

TRANSITIONING FROM BREAST TO BOTTLE

- The Mii™ teat will flatten to a natural ortho shape that conforms to the size and shape of baby's mouth during feeding to allow individualized memorization.
- The base of the teat has raised surface bumps that simulate the hardened “teething” surface that develops at the base of a woman's nipple due to repeated breast feeding, assisting in smooth breast to bottle transition when needed.
- During bottle feeding, babies often develop gas or colic symptoms as a result of poor ventilation.
- For this reason, Mii™ has specifically engineered the patent pending Flutter Flex Vent™ to help reduce the risk of those unwanted symptoms.
- The Flutter Flex Vent™ technology allows air to flow between the threads of the ring and neck, and into the spaces located at the top of the bottle neck.
- As baby feeds, the air pressure on the inside of the bottle becomes lower than the pressure on the outside of the bottle.
- This pressure difference causes the inner ring of the teat to flex inward, thereby allowing air to enter the bottle, and thus venting to occur.

THE RIGHT TEAT FOR YOUR BABY

- Flutter Flex Slow Flow Teat™
- Flutter Flex Medium Flow Teat™
- Flutter Flex Fast Flow Teat™
- Flutter Flex Y-Cut Teat™

TIPS FOR MOMS

TROUBLESHOOTING STEPS

- If you experience a lack of suction, make sure that the pump is properly and assembled and sealed.
- Check if the forked-end of the handle is fitted within one of the three settings on the stem, or move it further down the stem for stronger suction power.
- If you are unable to extract milk with the breast pump, try to relax, readjust your posture with your back straight, and reposition your breast properly into the suction cup.
- Sometimes the milk ducts in the breast become clogged if the breast is pressed too close to the suction cup.
- If you begin to experience pain, try adjusting the pump to a lesser suction strength by moving up on the stem settings.
- If problem persists, consult your professional health advisor or lactation consultant.

STORING BREAST MILK

- If you decide to store expressed breast milk, you should do so immediately using a proper milk bag or storage container.
- Be sure to only store milk that was collected with a sterile pump.
- There are many ways to store your breast milk:
- 5-8 days in a refrigerator, but do not use the shelf.
- 2 weeks inside of the freezer compartment of a refrigerator.
- 3-4 months in a refrigerator-freezer with a separate door.
- 6-12 months in a deep freezer with a temperature below -32° C (0° F).
- Freezing your breast milk is perfectly safe and will not compromise the nutrients or anti-bodies.

FEEDING YOUR BABY STORED MILK

- Breast milk should not be heated in a microwave or boiled because the special proteins and anti-bodies can be removed.
- Microwaving also causes uneven heating that can harm your baby.
- If you decide to microwave or boil for heating, please use extra care.
- If heating is necessary, place the storage container under running hot water until warm, or thaw by allowing to sit in a bowl of hot water.

BOTTLE FEEDING SAFETY CAUTIONS

- Please follow these safety precautions:
- Using a microwave oven is not recommended because of uneven heating and hot spots that could injure your baby.
- If you decide to use a microwave, always remove all other parts, including teat and ring and stir liquid contents to eliminate hot and cold spots.
- Test temperatures before feeding and do not warm liquid beyond body temperature 37° C (98.6° F)
- Use only under adult supervision; baby feeding parts are not toys.
- Do not allow your child to run or play with small parts.
- Never use bottle teats as pacifiers, soothers, or teethers.
- After feeding, remove the bottle from baby.
- Prolonged or continuous fluid sucking can cause tooth decay, even with milk and formula, so prolonged bottle use is not recommended.
- Use of sweetened or carbonated drink and fruit juices are not recommended.
- If using formula, avoid bubbles and do not shake the bottle.
- Formula should always be gently stirred to dissolve lumps that can clog teat holes.
- Never fill containers beyond the fill mark, as this can affect the venting system.
- Always assemble teat only when they are wet.
- Do not leave feeding objects and teats in direct sunlight.
- Keep teats in a dry, covered container when not in use.
- Prior to every use, inspect teats to make sure they are not torn or damaged from biting.
- Damaged teats may present a choking hazard.
- Discontinue use if any tear or crack appears.
- Replace Flutter Flex Teats™ every 8 to 12 weeks for hygienic reasons.
- Stop feeding before the teat is completely empty to reduce colic and digestive problems that can result from swallowing air.
- Always keep bottles, training and drinking cups in an upright position when transporting them.
- Do not insert pacifier cord through the holes used for ventilation.
- Clean brushes in hot soapy water or place in top rack of dishwasher, making sure to rinse thoroughly after every use.
- Allow to air dry.
- Always remove any debris or residue that might collect before assembly.

- Avoid over-tightening the bottle screw ring.
- Always ensure the pump parts are properly assembled.
- Keep your baby propped up during feeding.
- Use the Flutter Flex Teat™ only with Mii™ bottles.
- Contact your physician if you have any feeding questions or problems.

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