*No. 1 choice  
of hospitals  
& mothers*

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EN Instructions for use | RU Инструкция по применению  
HU Használati utasítás | CS Návod k použití  
TR Kullanım talimatları | PL Instrukcja obsługi

Thank you for choosing the Mini Electric Breastpump. Breastmilk is the natural choice for your baby, but there are reasons and situations where breastfeeding is not possible. For over 50 years, it has been Medela's conviction that these mothers and babies can be supported. In close cooperation with breastfeeding experts, Medela, one of the market leaders, has researched and developed breastpumps that perfectly meet breastfeeding mothers needs. For mothers who insist on the best for their babies and themselves. Because you care.

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## 13. EMC/Technical description

The Mini Electric breastpump needs special precautions regarding EMC (Electromagnetic compatibility) and needs to be installed and put into service according to the EMC information provided in these instructions for use. Wireless communications equipment such as wireless home network devices, mobile phones, cordless telephones and their base stations, walkie-talkies can affect the electric breastpump Mini Electric and should be kept at least a distance 1.0 m away from the equipment. Electromagnetic Compatibility (EMC, IEC 60601-1-2:2007, Table 1)

Electromagnetic emissions		
The electric breastpump Mini Electric is intended for use in the electromagnetic environment specified below. The customer or the user of the electric breastpump Mini Electric should assure that it is used in such an environment.		
Emission tests	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The electric breastpump Mini Electric uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Class A	The electric breastpump Mini Electric is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not applicable <sup>a</sup>	

<sup>a</sup> No significant voltage changes

**Warning** – The electric breastpump Mini Electric should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the electric breastpump Mini Electric should be observed to verify normal operation in the configuration in which it will be used.

Electromagnetic Compatibility (EMC, IEC 60601-1-2:2007, Table 2)

<b>Electromagnetic immunity</b>			
<p>The electric breastpump Mini Electric is intended for use in the electromagnetic environment specified below. The customer or the user of the electric breastpump Mini Electric should assure that it is used in such an environment.</p>			
<b>Immunity test</b>	<b>IEC 60601 test level</b>	<b>Compliance level</b>	<b>Electromagnetic environment – guidance</b>
Electrostatic discharge ( ESD ) IEC 61000-4-2	$\pm 6$ kV contact $\pm 8$ kV air	$\pm 6$ kV contact $\pm 8$ kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient / burst IEC 61000-4-4	$\pm 2$ kV for power supply lines $\pm 1$ kV for input / output lines	$\pm 2$ kV for power supply lines Not applicable <sup>a</sup>	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	$\pm 1$ kV line( s ) to line(s) $\pm 2$ kV line( s ) to earth	$\pm 1$ kV differential mode Not applicable <sup>b</sup>	Mains power quality should be that of a typical commercial or hospital environment.
<sup>a</sup> No input/output lines <sup>b</sup> No earth, class II equipment			

## 13. EMC/Technical description

<p>Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11</p>	<p>&lt;5 % <math>U_T</math> (&gt;95 % dip in <math>U_T</math>) for 0,5 cycle 40 % <math>U_T</math> (60 % dip in <math>U_T</math>) for 5 cycles 70 % <math>U_T</math> (30 % dip in <math>U_T</math>) for 25 cycles &lt;5 % <math>U_T</math> (&gt;95 % dip in <math>U_T</math>) for 5 s</p>	<p>&lt;5 % <math>U_T</math> (&gt;95 % dip in <math>U_T</math>) for 0,5 cycle 40 % <math>U_T</math> (60 % dip in <math>U_T</math>) for 5 cycles 70 % <math>U_T</math> (30 % dip in <math>U_T</math>) for 25 cycles &lt;5 % <math>U_T</math> (&gt;95 % dip in <math>U_T</math>) for 5 s</p>	<p>Mains power quality should be that of a typical commercial or hospital environment. If the user of the electric breastpump Mini Electric requires continued operation during power mains interruptions, it is recommended that the electric breastpump Mini Electric is powered from an uninterruptible power supply or a battery.</p>
<p>Power frequency (50/60 Hz) magnetic field IEC 61000-4-8</p>	<p>3 A/m</p>	<p>3 A/m</p>	<p>Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.</p>
<p>NOTE <math>U_T</math> is the a.c. mains voltage prior to application of the test level.</p>			

Electromagnetic Compatibility (EMC, IEC 60601-1-2:2007, Table 4)

<b>Electromagnetic immunity</b>			
<p>The electric breastpump Mini Electric is intended for use in the electromagnetic environment specified below. The customer or the user of the electric breastpump Mini Electric should assure that it is used in such an environment.</p>			
<b>Immunity test</b>	<b>IEC 60601 test level</b>	<b>Compliance level</b>	<b>Electromagnetic environment – guidance</b>
<p>Conducted RF IEC 61000-4-6</p>	<p>3 Vrms 150 kHz to 80 MHz</p>	<p>3 Vrms</p>	<p>Portable and mobile RF communications equipment should be used no closer to any part of the electric breastpump Mini Electric, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance <math>d = 1,2\sqrt{P}</math></p>
<p>Radiated RF IEC 61000-4-3</p>	<p>3 V/m 80 MHz to 2,5 GHz</p>	<p>3 V/m</p>	<p><math>d = 1,2\sqrt{P}</math> 80 MHz to 800 MHz <math>d = 2,3\sqrt{P}</math> 800 MHz to 2,5 GHz where <math>P</math> is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and <math>d</math> is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,<sup>a</sup> should be less than the compliance level in each frequency range.<sup>b</sup> Interference may occur in the vicinity of equipment marked with the following symbol: ((Ⓜ))</p>

## 13. EMC/Technical description

Note 1 At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

<sup>a</sup> Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the electric breastpump Mini Electric is used exceeds the applicable RF compliance level above, the electric breastpump Mini Electric should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the electric breastpump Mini Electric.

<sup>b</sup> Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Electromagnetic Compatibility (EMC, IEC 60601-1-2:2007, Table 6)

**Recommended separation distances between portable and mobile RF communications equipment and the electric breastpump Mini Electric**

The electric breastpump Mini Electric is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the electric breastpump Mini Electric can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the electric breastpump Mini Electric as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter M		
	150 kHz to 80 MHz $d = 1,2\sqrt{P}$	80 MHz to 800 MHz $d = 1,2\sqrt{P}$	800 MHz to 2,5 GHz $d = 2,3\sqrt{P}$
0,01	0,12	0,12	0,23
0,1	0,37	0,37	0,74
1	1,2	1,2	2,3
10	3,7	3,7	7,4
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance  $d$  in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

Note 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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Medela AG