## FingerFeeder

Breastmilk is the best nutrition for babies. Sometimes however, situations arise, in which breastfeeding is not possible momentarily, e.g. babies with feeding difficulties (babies with Down syndrome, cleft lip/palate or premature babies after the 36th week of pregnancy). Breastmilk can be given to these babies in small amounts via syringe.

**Finger feeding should be done under the supervision of a professional (lactation consultant IBCLC), as it requires exact knowledge of the possibilities and risks, as well as the precise following of these instructions. If anything is not clear, especially trained staff who is familiar with this feeding method should be consulted.**

### Use

Wash your hands thoroughly and make sure that your fingernails are short before starting to finger feed. Hold the baby slightly upright or swaddled in a cloth on your lap according to the instruction of the professional. A nursing pillow or nursing stool can be used for comfortable sitting.

The FingerFeeder is then placed on a syringe (10 or 20 ml). Fill the syringe with the prescribed amount of breastmilk or supplement recommended by your doctor. Slide the little or index finger into the mouth of the baby, so the pad side touches the baby's hard upper palate. Slowly slide the finger to the soft palate. The fingernail should be resting on the baby's tongue. Do this carefully, as not to trigger the gag reflex. Normally the baby will begin sucking when the finger pad touches the hard palate. Then you can slide the FingerFeeder carefully next to the finger in the corner of the baby's mouth – to about the middle of the first finger joint.

When the baby starts sucking, you will feel a sucking sensation on the finger nail and your finger will be pulled further into the baby's mouth. As soon as the baby swallows, the pressure will be reduced. The sucking and swallowing should be in a rhythmic pattern. We recommend to note the amount of milk that the baby receives with each finger feeding. The amount of milk filled into the syringe should be according to the doctor or professional's prescription. Sometimes the baby will not drink the entire amount. However, it is still important to fill the syringe to this amount in order to avoid air swallowing on an empty syringe.

### Cleaning and Sterilization

Before first use and after every use, the FingerFeeder must be cleaned and sterilized. This can be done by boiling in water for 5 minutes, by autoclaving (at 134°C for 3 (or 18) minutes or at 125°C for 20 minutes).

### Technical Data

Environmental condition for transport and storage:  
 – Temperature: –20°C to + 50°C  
 – Relative humidity: 20% to 95%  
 – Atmospheric Pressure: 70 kPa to 106 kPa

### Disposal

According to local regulations.  
 CE-mark according to 93/42/EEC

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