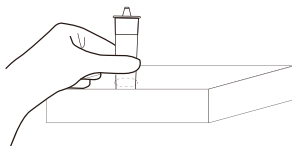
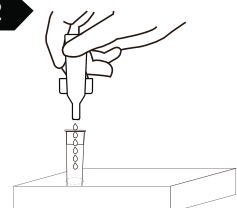
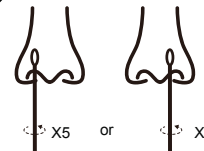
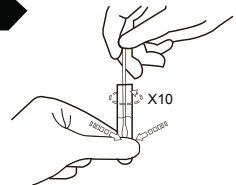
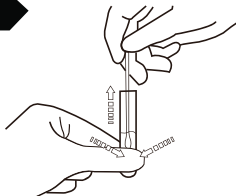
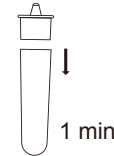

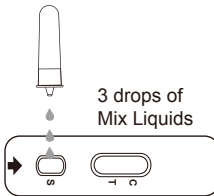
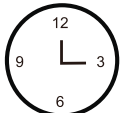
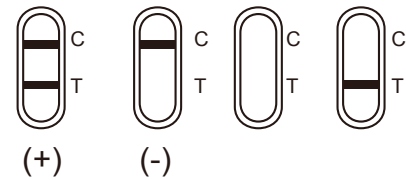
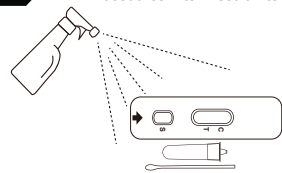




Singclean® COVID-19 Test Kit Self-Testing Procedure

<p>1</p>  <p>The 1 test box has a hole on the surface and then can be used as a workbench. Put the extraction tube in to the hole.</p>	<p>2</p>  <p>Add 6 drops of extraction buffer.</p>	<p>3</p>  <p>keep the head slightly tilted, and gently and slowly insert the swab through the nasal cavity (about 2-4cm), rotate gently 5 times, and gently rotate to remove the swab.</p>	<p>4</p>  <p>Insert the swab into an extraction tube, rotate the swab about 10 times while pressing the swab head against the tube wall to release the antigen in the swab. let it stand for about 1 minute.</p>	<p>5</p>  <p>Remove the swab while squeezing the tip of the swab to extract the liquid from the swab.</p>	<p>6</p>  <p>Install the dropper on the extraction tube and cap it tightly, and let it stand for about 1 minute.</p>	<p>7</p>  <p>Open the aluminum foil bag and take out the test card.</p>
<p>8</p>  <p>Apply 3 drops of extracted solution to the specimen well of the test device.</p>	<p>9</p>  <p>Wait for the colored line to appear. The result should be read in 15 minutes. Do not interpret the result after 20 minutes.</p>	<p>10</p>  <p>Positive (-) Negative (-) Invalid</p>	<p>11 Procedures After Result Interpretation</p>  <p>Use a household bleach spray, or a 70% - 75% alcohol spray to disinfect used product components.</p>  <p>Put used product components in a plastic bag. Close the bag and put it in another plastic bag. Dispose of the bag in accordance with biohazard waste disposal methods.</p>  <p>Wash the hands thoroughly with soap or use hand sanitizer.</p>			