



# out now! AS UNIQUE AS YOU

[www.mysize-condoms.com](http://www.mysize-condoms.com)

## It could be more than you expected.

### Everything about the condom:

Easy to open, pleasant scent, 9 widths, ultra thin. Low allergen **vytex** latex, vegan, produced in fair conditions for people and the environment. Ultra-sensitive and safe.

You'll forget you're even wearing a condom.



**NOTE:** Penis circumference determines the recommended width for **MY.SIZE PRO** - not penis length.

**IMPORTANT:** Your sensitivity levels are determined by different condom widths. It's important that the condom doesn't slip off or that it is too tight.



### IMPORTANT:

When printing, select „NO“ page adjustment. Otherwise the measuring tape will not be printed out with the correct dimensions. Download the file and print it out with Adobe Acrobat **WITHOUT** page adjustment.

Spread happiness with **MY.SIZE PRO**: cut off the upper strip for a friend.

## Step by step. That's how easy it is.

**01.** Cut off the **MY.SIZE PRO** measuring tape on the dotted line.




**02.** Carefully place the measuring tape on the widest part of the erect penis. See the recommended width at the arrow.

**IMPORTANT:** Be careful of sharp paper edges when cutting off the tape!



**03.** An initial recommendation - try out different sizes and this width.

It's about your sensitivity levels - and what you want.

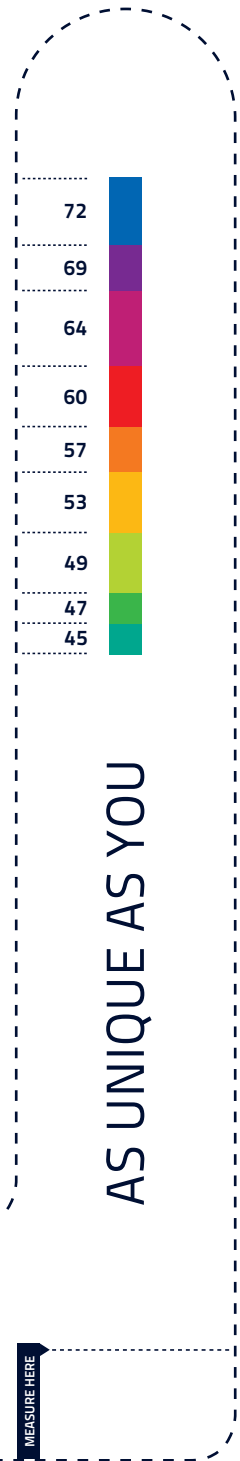

MEASURE HERE